BT60

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BT60

Dual Spore Strips Biological Indicator

For Ethylene Oxide (EO), Steam and Dry Heat sterilization processes.

Usage

Monitoring Ethylene Oxide, Steam, and Dry Heat Sterilization Processes.

Applicable Regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.

ISO 11138-1:2017, ISO 11138-2:2017, ISO 11138-3:2017 and ISO 11138-4:2017; IRAM 37102-1:1999, IRAM 37102-2:1999 and IRAM 37102-3:1999.

Classification

Class 1, according to risk.

Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-1.

Characteristics

25 mm x 70 mm non-absorbent material package, Violet code.

5 mm x 25 mm filter paper strips.

10^5 or 10^6 *Geobacillus stearothermophilus* ATCC 7953 spores and 10^6 *Bacillus atrophaeus* ATCC 9372 spores per strip.

Upon completion of the sterilization cycle, the spore strip should be tested in a suitable culture medium at 37 °C or at 60 °C. We suggest the use of BIONOVA® MC1020 or MC1030 culture media to final reading in 48 hours at 37 °C for Ethylene Oxide and Dry Heat sterilization process control, or BIONOVA® MC20 or MC1020 culture media to final reading in 24 hours at 60 °C for Steam sterilization process control. For conventional culture media like TSB, incubation for 7 days is recommended.

D-Value_{STEAM}: Not lower than 1.5 minutes. Conditions: 121 °C.

Z-Value_{STEAM}: Not lower than 6 °C.

D-Value_{EO}: Not lower than 2.0 minutes. Conditions: 600 mg/l EO, 60 % RH, 54 °C.

D-Value_{DRY HEAT}: Not lower than 2.0 minutes. Conditions: 160 °C.

Z-Value_{DRY HEAT}: Not lower than 20 °C.

Environmental conditions during manufacture

Room temperature 15-30 °C, RH 30-80 %. Sterility conditions are necessary only during the inoculation process performed in laminar flow.

Storage conditions

T = 10-30 °C, RH 30-80 %.

Transportation conditions

Storage conditions should be strictly followed.

Products should be transported in closed and reinforced boxes in order to avoid damages. Product transportation does not represent any risk for human health.

Shelf-life

2 years.

Packing

100 units per bag.

Packing information: product code and description, process for intended use, regulation, storage conditions, manufacturer information.

Information on product's package: product code and description, process for intended use, bacterial strain, batch number, expiration date and datamatrix code.

Labelling

On product's bag: product code and description, bacterial strain and load, batch number, manufacture date and expiration date, barcode and datamatrix code.

Possible target markets

Healthcare and Industry

Other important information

It is advisable to incubate at 37 °C or at 60 °C in Bionova® IC10/20 incubator.

Read product's instructions for use thoroughly before use.

Description of identified and/or predictable risks which could represent a risk for the product

Do not store the product near sterilizing agents.

Do not expose this product to radiation, H₂O₂ or any sterilization process other than EO, Steam and/or Dry Heat.

The positive biological indicator can be autoclaved at 121 °C for at least 20 minutes, at 132 °C for at least 15 minutes in gravity displacement steam sterilization, or at 134 °C for at least 10 minutes in vacuum steam sterilizer